


Charger overview 2024

Charger options

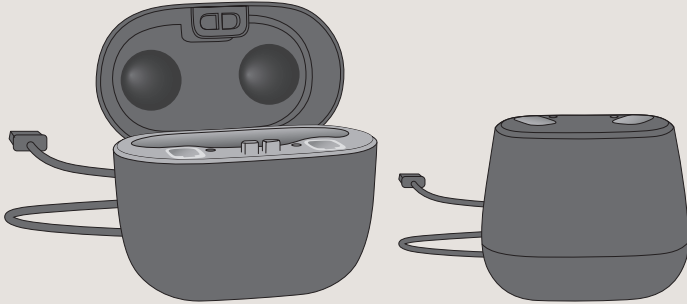


SmartCharger
(Portable charger)

Charger

Hearing aid charging times

Charging time for hearing aids



Charging time


15 min	30 min	60 min	120 min
4 hrs	8 hrs	16 hrs	20 hrs

Use time*

SmartCharger charging times

Charging time for power bank battery	Charging time for power bank + hearing aids
Up to 4 hrs	Up to 6 hrs
Fully charged	Fully charged

SmartCharger battery indicator

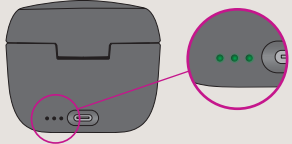

LED lights

SmartCharger connected to power source	SmartCharger disconnected (Power bank battery level)
Charging - Low battery	70-100%
Charging	40-70%
Fully charged	Less than 40%

Care for SmartCharger & Charger

Keep charger dry & clean with a dry cloth only	Optimal charging temperatures are between +15°C and +35°C (+59°F and +95°F)	Chargers can be powered from your car or computer using a USB cable if the socket is USB 2.0 compliant, and has a minimum of 500mA output
--	---	---

* Expected use time for rechargeable battery depends on use pattern, active feature set, hearing loss, sound environment, battery age and use of wireless accessories.