


Charger overview 2024

Charger options

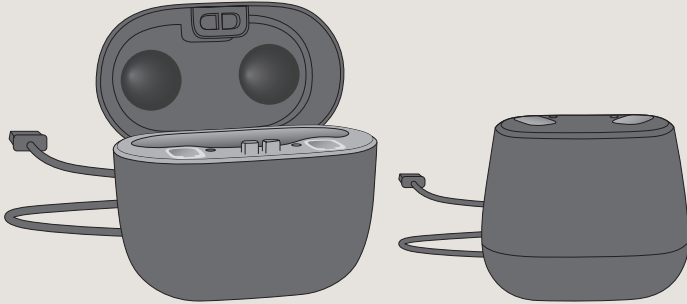


SmartCharger
(Portable charger)

Charger

Hearing aid charging times

Charging time for hearing aids



Charging time


15 min	30 min	60 min	120 min
4 hrs	8 hrs	16 hrs	20 hrs

Use time*

SmartCharger charging times

Charging time for power bank battery	Charging time for power bank + hearing aids
Up to 4 hrs	Up to 6 hrs
Fully charged	Fully charged

SmartCharger battery indicator

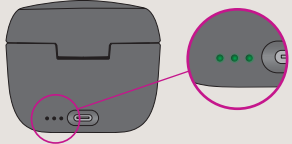

LED lights

SmartCharger connected to power source	SmartCharger disconnected (Power bank battery level)
Charging - Low battery	70-100%
Charging	40-70%
Fully charged	Less than 40%

Care for SmartCharger & Charger

Keep charger dry & clean with a dry cloth only	Optimal charging temperatures are between +15°C and +35°C (+59°F and +95°F)	Chargers can be powered from your car or computer using a USB cable if the socket is USB 2.0 compliant, and has a minimum of 500mA output
--	---	---

* Expected use time for rechargeable battery depends on use pattern, active feature set, hearing loss, sound environment, battery age and use of wireless accessories. Performance of the hearing devices is dependent on individual circumstances and may not be appropriate for all conditions. Always read the label and follow the instructions.