## Echoport 292

#### Advanced clinical binaural OAEs



## Australian Distributor

SONIC EQUIPMENT One stop shop for all your audiological needs

1800 639 263 info@soniceq.com soniceq.com











# Echoport 292

#### Advanced clinical binaural OAEs

Satisfies the most demanding of clinical needs. Providing a range of Gold Standard clinical OAE tests.



**POWERFUL ANALYSIS** Clinical level DPOAE, TEOAE, SOAE



FULLY CUSTOMIZABLE 4 Protocols Up to 14 parameters



FREQUENCY RANGE DPOAE 0.8 to 10.5kHz TEOAE 1 to 6kHz PASS SNR 1 to 14dB per 1/2 octave ( $\Delta$ 1dB)







2

- · Robust, compact format
- USB plug-n-play connection to PC
- Lightweight and battery-free (powered by USB connection)

OAE PROBE

- Windows 10, 8 and 7 compatible
- Full spectrum display during DP test shows higher order DPs
- Contralateral noise for suppression of TEOAE
- Powerful 'synchronized' spontaneous emission search to reveal actual and latent spontaneous OAEs

	DP <b>OAE</b> /	TE <b>OAE</b> /
Test Configurability	4 Protocols, 14 Parameters	4 Protocols, 10 Parameters
Frequency Range	0.8 to 10.5 kHz	1 to 6 kHz, 100 pts
Freq. Resolution	1,2,3,4,8 pts/octave	50Hz, 80Hz or 1/2 octave
Stimulus Levels	40 to 80 (∆1dB) (dBSPL)	40 to 90 (A1db) (dBSPLpe)
Pass Criteria	10 Parameters	10 Parameters
Pass SNR	1 to 14dB per 1/2 octave ( $\Delta$ 1dB)	1 to 14dB per 1/2 octave ( $\Delta$ 1dB)







- High resolution DPgram up to 8 points per octave (27 frequencies shown)
- Preset and customizable stimuli
- Programmable L1 and L2 level, frequency ratio, points/octave, frequency range, manual/automatic/intelligent frequency progression
- Total DP power assessment
- DP I/O growth rate analysis, dB/dB or scissor optimized stimuli
- Simultaneous binaural recording (ILO292-II only)
- True time domain averaging and noise statistics
- Ultra secure SNR assessment
- Total DP power assessment



## TEOAE RECORDING

- Quick Screen and Standard 'ILO' (20ms) diagnostic recording
- Click stimulus or configurable tone pulse
- Configurable stimulus intensity. In-the-ear calibration
- Real-time stimulus and probe fit monitoring
- 'ILO' analysis giving OAE dBSPL and reproducibility in half-octave bands plus stimulus and response waveforms
- Simultaneous binaural recording (ILO292-II only)



## TWO MODELS

## 

### **Echoport** 292 **I**

The powerful Echoport 292-I satisfies the most demanding of clinical needs. Providing a range of Gold Standard clinical OAE tests TEOAE, DPOAE, SOAE, DP Growth.



### Echoport 292

The Echoport 292 II dual probe delivers all the power of the Echoport 292 I and adds bilateral TE&DP recording plus Contralateral Suppression



## ANALYSIS TOOLS

- Examine/compare feature for DP-DP, TE-DP and TE-TE
- Full cross correlation, frequency analysis with reproducibility and signal-to-noise data on single test or between test pairs
- · Waveform analysis with response subtraction
- Detailed TEOAE spectrum analysis (50Hz resolution).
- · Half-octave OAE power assessment
- Full numerical OAE data table provided



- · Fully searchable patient database with printable reports
- ILO standard DTA and DPG files
- Exportable data to OZ, Hi\*Track, XML, ILOV5, Excel and ASCII (user selectable data elements)
- Import of data from other V6, Otolink or Ez-screen sites
- · Integration of data from multiple sites, testers and machines
- · Ability to run from a network and with a network databases

Clinical Audiology







L2 Stm OP Leve

For further information, please contact us

www.otodynamics.com





- OAE Probes
- USB cables 2m & 0.5m
- · Test cavity
- Startup probe tip box
- V6 & Ez-screen PC software
- Carry case







### Australian Distributor



1800 639 263 info@soniceq.com soniceq.com

#### Otodynamics Ltd.

30-38 Beaconsfield Road Hatfield Herts AL10 8BB UK т. 1-800-659-7776 е. info@otodynamics.com

www.otodynamics.com

Ref: F-008-12USA iss001